


Preferred Prep Guide

Anterior Chairside Preparation Guide


Full-Coverage Restorations

ANTERIOR CROWN PREPARATION

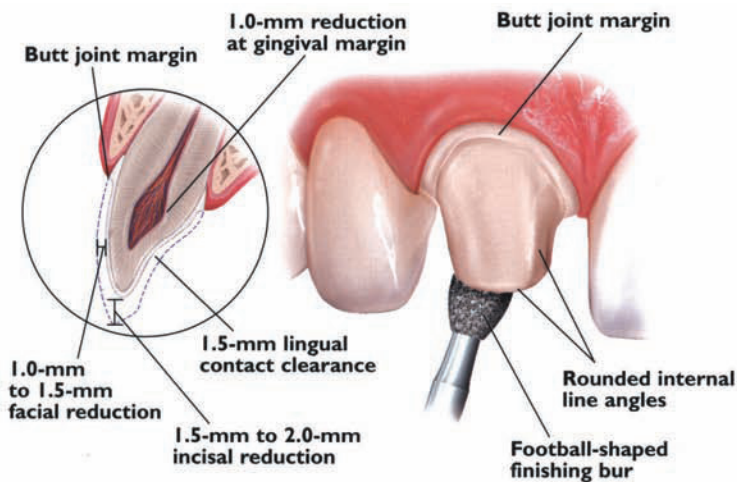

3-Unit Bridge Restorations


3-UNIT BRIDGE PREPARATION

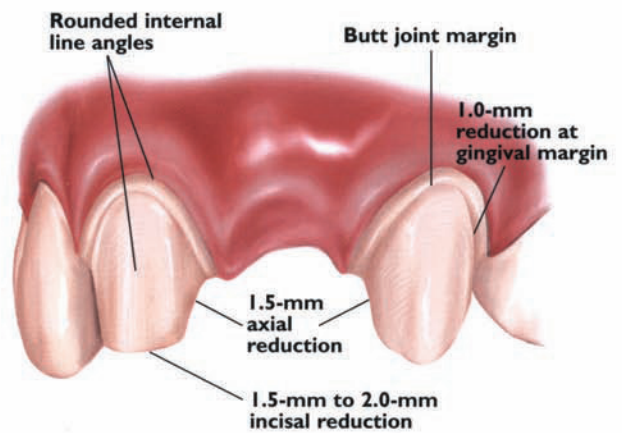

Veneers


VENEER PREPARATION

medium grit, round-ended, diamond bur is used to remove a uniform thickness of facial enamel by joining the depth-cut grooves.

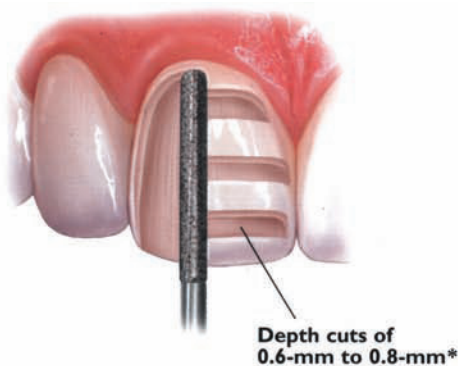


INCISAL PREPARATION

The diamond bur is angled to bevel back the incisal edge.

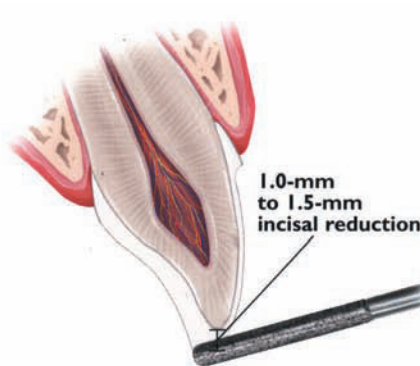

CHAMFER MARGINS

Correct preparation of the chamfer margins interproximally allows the appropriate bulk of porcelain.

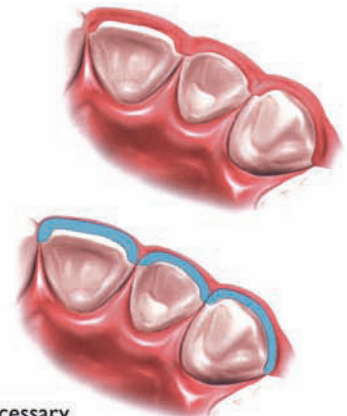

*Please note: For additional masking capabilities and/or layering techniques, further reduction may be necessary. Contact your laboratory for further information.